MEDICAL DEVICE TECHNOLOGIES, INC.
3600 S.W. 47th Avenue Gainesville, Florida 32608 USA
Domestic: 877.991.1110; International 732.626.6466
email: customer_service@angio.com
www.angiotech.com


STERILE EO

EC REP PBN Medicals, Denmark A/S,
Knud Bro Alle 3, DK-3660 Stenlose, Denmark

ENG: PRO-MAG™ BIOPSY NEEDLE
SPA: AGUJA PARA BIOPSIA PRO-MAG™
FRE: AIGUILLE DE BIOPSIE PRO-MAG™
POR: AGULHA DE BIOPSIA PRO-MAG™
GER: PRO-MAG™ BIOPSIENADEL
DUT: PRO-MAG™ BIOPSIENAALD
ITA: AGO DA BIOPSIA PRO-MAG™
SWE: PRO-MAG™ BIOPSIÑAL
GRE: ΒΕΛΟΝΑ ΒΙΟΨΙΑΣ PRO-MAG™
FIN: PRO-MAG™-BIOPSIANEULA
DAN: PRO-MAG™ BIOPSIÑAL
NOR: PRO-MAG™ BIOPSIÑAL
TUR: PRO-MAG™ BIYOPSI İGNESİ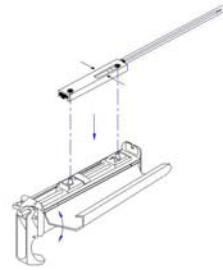

ENGLISH

Description: The Pro-Mag™ Biopsy Needle is used in conjunction with a Pro-Mag™ Automatic Biopsy Instrument to harvest soft tissue core biopsies.

Indications for Use: The Pro-Mag™ Biopsy Needle is used to obtain multiple core samples from soft tissue such as the liver, kidney, prostate, breast, etc; and is used in conjunction with the Pro-Mag™ Automatic Biopsy Instrument.

Contraindications: For use only for core biopsies of soft tissue as determined by a licensed physician. The instrument should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of core needle biopsy. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

Directions For Use:

1. Inspect the product package for damage. If undamaged, open the package using proper aseptic technique.
2. Remove the needle from the pouch. Inspect the stylet for damage to the point, the cannula for damage to the cutting edge, or other imperfections that would prevent proper operation of the needle assembly.
3. Open the lid of the biopsy device, load needle into the biopsy device, close lid, and remove protective sheath.
4. Cock the actuator handle twice in order to activate the device for firing (this automatically sets the safety.)
5. Release the safety by pushing the "Safety Release Button" located on the side of the device.
6. Firing the device can be achieved by pushing in on either the front trigger button or the rear trigger button. This action will trigger the spring mechanism, propelling the needle forward for a core biopsy sample.

NOTE: The needle need not be removed from the biopsy device to retrieve the sample.

7. After removing the needle from the patient, cock the actuator handle once to expose the biopsy notch located at the distal end of the needle. This action exposes the specimen for removal. Cocking the actuator handle a second time will activate the device for a second biopsy.

NOTE: If the needle is to be inserted into patient prior to loading into biopsy instrument, squeeze spacer in slot area (as indicated) to prevent cannula hub movement.

WARNING: This disposable needle set is for Single Patient Use Only. Do not attempt to clean or resterilize the needle set. After use this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

Pro-Mag™ is a trademark of Medical Devices Technologies, Inc.

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Pro•Mag™ er et varemerke som tilhører Medical Device Technologies, Inc.
Pro•Mag™ Medical Device Technologies, Inc.'nin ticari markasıdır.