**MEDICAL DEVICE TECHNOLOGIES, INC.**  
3600 S.W. 47th Avenue Gainesville, Florida 32608 USA  
Domestic: 877.991.1110; International 732.626.6466  
email: customer\_service@angio.com  
www.angiotech.com


STERILE EO

EC REP

PBN Medicals, Denmark A/S,  
Knud Bro Alle 3, DK-3660 Stenlose, Denmark

Eng: Automatic Cutting Needle For Bard® Magnum™ Biopsy Instrument  
Spa: Aguja De Corte Automático Para El Instrumento De Biopsias Bard® Magnum™  
Fre: Aiguille Auto-Coupante Pour Dispositif De Biopsie Bard® Magnum™  
Por: Agulha Cortante Automática Para Instrumento De Biópsias Bard® Magnum™  
Ger: Automatische Stanznadel Für Das Bard® Magnum™ Biopsieinstrument  
Dut: Automatische Snijnaald Voor Het Bard® Magnum™ Biopsie-Instrument  
Ita: Ago Tagliente Automatico Per Strumento Per Biopsia Bard® Magnum™  
Swe: Automatisk Skärnål För Bard® Magnum™ Biopsiinstrument  
Gre: Αυτόματη Βελόνα Κοψίματος Για Το Εργαλείο Βιοψίας Bard® Magnum™  
Fin: Automaattinen Leikkausneula Bard® Magnum™ Biopsiinstrumenttiin  
Dan: Automatisk Biopsi Kanyle Til Bard® Magnum™ Biopsi Instrument  
Nor: Automatisk Skjærenål For Bard® Biopsi-Instrument  
Tur: Bard® Magnum™ Biyopsi Aleti İçin Otomatik Kesici İğne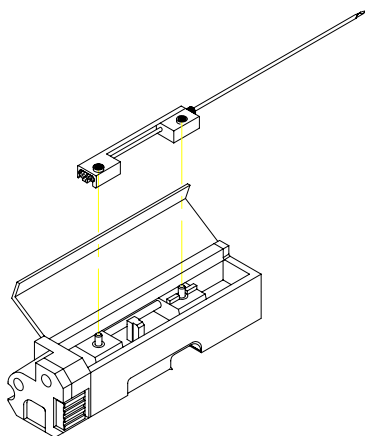

## ENGLISH

**Indications for Use:** This Automatic Cutting Needle is intended for use as a Core Biopsy Needle and is to be used in conjunction with the Bard® Magnum™ Biopsy Instrument or other compatible, commercially available biopsy instruments.

**Contraindications:** For use only for core biopsies of soft tissue as determined by a licensed physician. The instrument should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of core needle biopsy. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

### Directions for Use:

1. Inspect the product package for damage. If undamaged, open the package using proper aseptic technique.
2. Remove the needle from the pouch and remove the protective tip protector. Inspect the stylet for damage to the point, the cannula for damage to the cutting edge, or other imperfections that would prevent proper operation of the needle assembly.
3. Open the lid of the biopsy device and load the needle set into the device as illustrated. Needle Set may be loaded in either cocked or uncocked position.
4. Operate the biopsy device following the manufacturer's operating instructions.

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