MEDICAL DEVICE TECHNOLOGIES, INC.  
3600 S.W. 47th Avenue Gainesville, Florida 32608 USA  
Domestic: 877.991.1110; International 732.626.6466  
email: customer\_service@angio.com  
www.angiotech.com

STERILE EO

PBN Medicals, Denmark A/S,  
Knud Bro Alle 3, DK-3660 Stenlose, Denmark

Eng: Pro•Mag™ Ultra Automatic Biopsy Instrument  
Spa: Instrumento para Biopsia Automático Pro•Mag™ Ultra  
Fre: Instrument de Biopsie Automatique Pro•Mag™ Ultra  
Por: Instrumento de Biopsia Automática Pro•Mag™ Ultra  
Ger: Pro•Mag™ Ultra Automatischen Biopsieinstrument  
Dut: Pro•Mag™ Ultra Automatisch Biopsie-instrument  
Ita: Pistola Bioptica Automatica Pro•Mag™ Ultra  
Swe: Pro•Mag™ Ultra Automatiskt Biopsiinstrument  
Gre: Αυτόματο Όργανο Βιοψίας Pro•Mag™ Ultra  
Fin: Automaattinen Pro•Mag™ Ultra -Biopsiinstrumentti  
Dan: Pro•Mag™ Ultra Automatisk Biopsiinstrument  
Nor: Pro•Mag™ Ultra Automatisk Biopsiinstrument  
Tur: Pro•Mag™ Ultra Otomatik Biyopsi Aleti

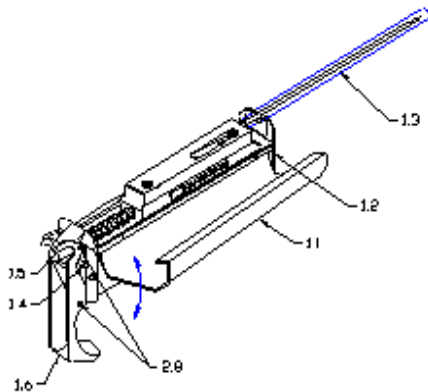
Figure 1 / Figura 1 / Figure 1 / Figura 1 / Abb. 1 / Figuur 1 /  
Figura 1 / Fig. 1 / Σχήμα 1 / Kuva 1 / Fig. 1 / Figur 1 / Şekil 1

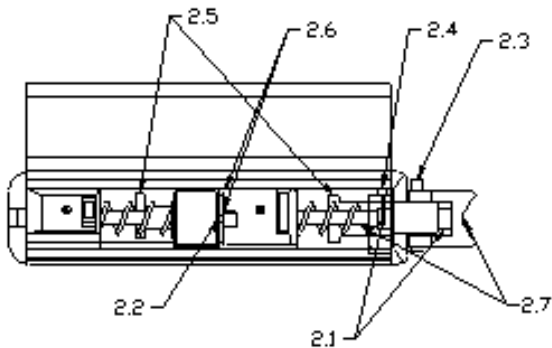
Figure 2 / Figura 2 / Figure 2 / Figura 2 / Abb. 2 / Figuur 2 /  
Figura 2 / Fig. 2 / Σχήμα 2 / Kuva 2 / Fig. 2 / Figur 2 / Şekil 2

## ENGLISH

**Description:** The Pro•Mag™ Ultra Automatic Biopsy Instrument is an automatic, reusable system for histological core biopsies. Pro•Mag™ Ultra is available in two models:

- Pro•Mag™ Ultra has a throw (advancement) of 25mm.
- Pro•Mag™ Ultra ST has a throw (advancement) of 14mm.

**Indications for Use:** The Pro•Mag™ Ultra Automatic Biopsy Instrument is used to obtain tissue core biopsy in the following clinical areas:

- **Urology** - Transrectal or transperineal biopsy of the prostate.
- **Radiology** - Percutaneous kidney, liver, lung and breast biopsy.

**Contraindications:** For use only for core biopsies of soft tissue as determined by a licensed physician. The instrument should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of core needle biopsy. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

**Note:** These instructions for the Pro•Mag™ Ultra Automatic Biopsy Instrument are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

**Caution:** The Pro•Mag™ Ultra Automatic Biopsy Instrument should be used by or under the supervision of a physician who is completely familiar with the use and limitations of this device and biopsy sampling.

### Directions For Use:

1. Inspect the disposable needle set package for damage. If undamaged, open the package using proper aseptic technique.
2. Remove the needle (1.3) from the pouch. Inspect the stylet for damage to the point, the cannula for damage to the cutting edge, or other imperfections that would prevent proper operation of the needle assembly.
3. Open the lid (1.1) of the biopsy device, load needle set (1.3) into the biopsy device, close lid (1.1), and remove protective sheath.
4. Safety button (2.3) must be set in the position indicated by the arrow on the lid before cocking.
5. Cock the actuator handle (1.6) twice in order to activate the device for firing (this automatically sets the safety.)
6. Release the safety by pushing the "Safety Release Button" (2.3) located on the side of the device.
7. Firing the device can be achieved by pushing in on either the front trigger button (1.2) or the rear trigger button (1.5). This action will trigger the spring mechanism, propelling the needle forward for a core biopsy sample.
8. After removing the needle from the patient, reset safety button (2.3) and cock the actuator handle once to expose the biopsy notch located at the distal end of the needle. This action exposes the specimen for removal. Cocking the actuator handle a second time will activate the device for a second biopsy.

**NOTE:** The disposable needle set is for Single Patient Use Only. Do not attempt to clean or re-sterilize the needle set. After use this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

### REPROCESSING INSTRUCTIONS FOR THE PRO•MAG™ ULTRA AUTOMATIC BIOPSY SYSTEM (Per ISO 17664-2004):

<b>Warnings / Precautions</b>	<ul style="list-style-type: none"> <li>• Never place the device in bleach or bleach solution.</li> <li>• Lubrication on the sides of the carrier blocks will slow the action of the device</li> </ul>			
<b>Limitations</b>	After cleaning and sterilization, verify functionality prior to re-use.			
<b>INSTRUCTIONS</b>				
<b>Point of Use</b>	This product is provided non-sterile and must be cleaned and sterilized before using.			
<b>Containment and Transportation</b>	No particular requirements			
<b>Preparation for cleaning</b>	No particular requirements. Ultrasonic treatment of the device has not been tested.			
<b>Cleaning: Automated</b>	<p>The Pro•Mag™ device has been validated for effective cleaning using an automated washer/disinfector cycle of a minimum of 30 minutes total time, including a pre-wash, enzyme wash, main wash, rinse and dry.</p> <p>Immerse instrument, with lid open, in the washer-disinfector and refer to the manufacturer's recommendations and instructions. Examine device for any residual soil. Thoroughly dry the device or it will not function properly.</p>			
<b>Cleaning: Manual</b>	<p>The Pro•Mag™ device should be cleaned in a mild detergent (e.g. ENZOL Enzymatic Detergent) and lukewarm water solution, per manufacturer's recommendations.</p> <p>Immerse instrument, with lid open, into solution and use a soft brush to clean any residue left from the procedure. After washing completely, rinse the device in hot water. Thoroughly dry the device or it will not function properly.</p>			
<b>Disinfection</b>	A high level of disinfection can be achieved with CIDEX® or ProCide NS; follow the manufacturer's recommended procedure. Thoroughly dry the device or it will not function properly.			
<b>Packaging</b>	A standard, sterilization wrap may be used. In the US, an FDA approved surgical wrap must be used.			
<b>Sterilization (Temperatures are minimum required, times are minimum required)</b>	The instrument should first be cleaned, lubricated and packaged prior to sterilization.			
	<b>Cycle:</b>	<b>Gravity</b>	<b>Gravity</b>	<b>Pre-vac</b>
	<b>Temp.:</b>	250° F (121°C)	270° F (132°C)	270° F (132°C)
	<b>Time:</b>	40 - 50 min	10 - 25 min	4 min
	<b>Drying:</b>	20 min, or until visibly dry		
<b>Maintenance, Inspection and Testing</b>	Washing detergents may eventually remove the lubricant originally applied. Lubricate device with silicone free, steam sterilization compatible lubrication (e.g., Aesculap® Sterilit® I Oilspray JG 600) prior to sterilization. Refer to the manufacturer's instructions for the use of the selected lubricating agent. Lubricate at the following points as displayed on Figure 2 :			

	2.1 Trigger Button (Must move freely) 2.2 Top End of Actuator Pin 2.3 Safety Button (Must move freely; lube both sides) 2.4 Lever Pivot 2.5 Top Edge of Ramps on Latch 2.6 Sides of Catch on Stylet Holder Block 2.7 Cam Surfaces 2.9 Link Pivots (Shown on Figure 1)  After cleaning and sterilization, verify functionality prior to re-use.
<b>Storage</b>	No specific requirements
<b>Additional Information</b>	None

Note: The instructions provided above have been validated by the manufacturer of the medical device as being CAPABLE of preparing a medical device for re-use. It remains the responsibility of the processor to ensure that the reprocessing as actually performed using equipment, materials and personnel in the reprocessing facility achieves the desired result. This normally requires validation and routine monitoring of the process.

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